

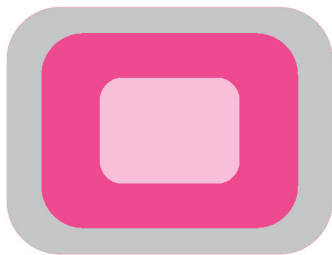


# CLINICAL TRAINING AIDS

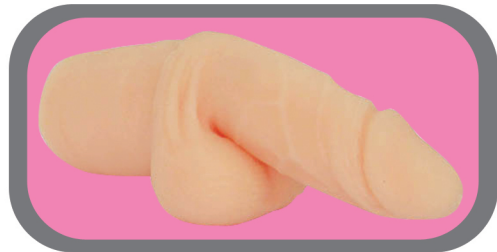

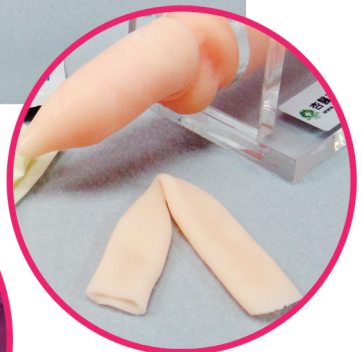

**GM-SP0060**  
**Circumcision Training Set**


This training set is designed for practicing surgical skill and different modes of circumcision on a child or a baby. Each set contains one child circumcision model or one baby circumcision model, one pack of foreskin piece, one circumcision holder and a pack of necessary accessories to start practice. Come with a disc of demonstration video.

Content of Accessories:


1. Disposable glove (1)
2. Dry powder (1)
3. Alcohol pad (1)
4. 45 cm suture with needle (1)
5. Plastibell (1)

## Instruction for foreskin replacement of the circumcision model


1. Prepare a circumcision model and a piece of foreskin replacement,

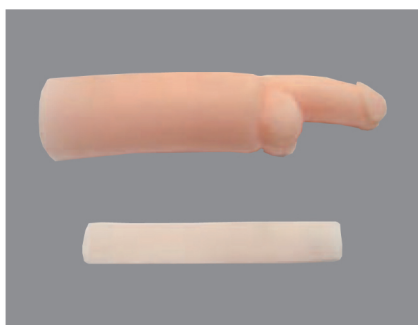


2. Mount one end of the foreskin onto the circumcision model.

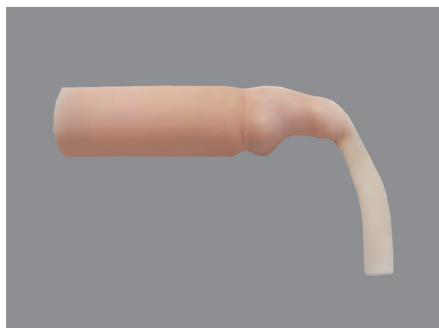


3. Furl and mount the other end of the foreskin.

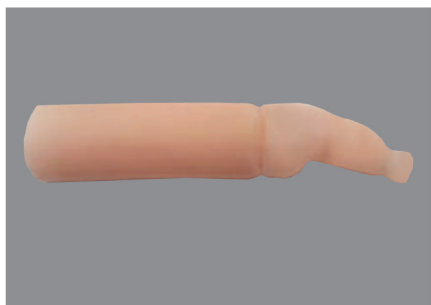

4. Mount the model into the holder stand, and prepare to perform circumcision.

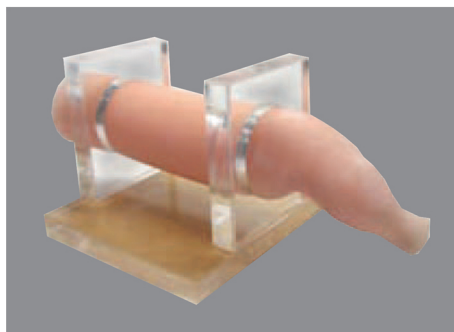


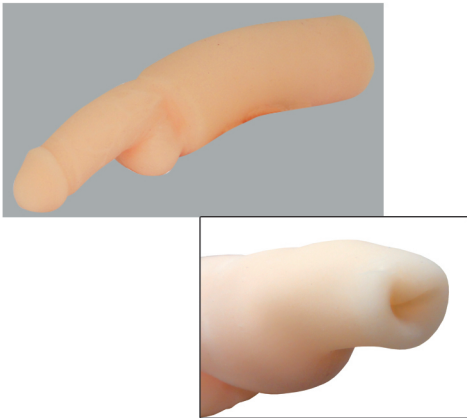

**GM-SP0061**

**Child Circumcision Model**


The model is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable, and has a shape and size similar to those of a 6~9 years-old boy. It has a realistic tactile feeling and is suitable for practicing a child circumcision.

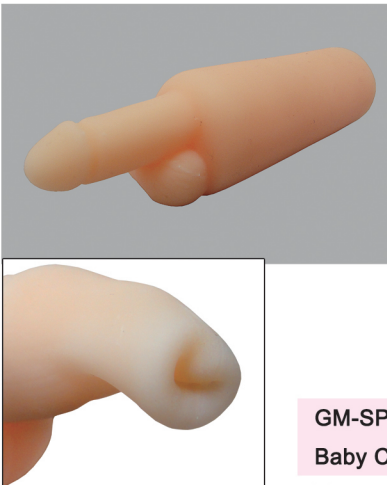

**GM-SP0063**

**Baby Circumcision Model**


The model is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable, and has a shape and size similar to those of a baby boy. It has a realistic tactile feeling and is suitable for practicing a baby circumcision.

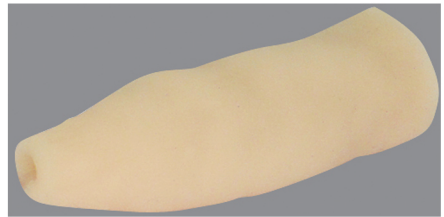

**GM-SP0062**

**Foreskin Piece Pack**

The tube-like piece is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable. It is disposable and used for foreskin replacement after each circumcision. Each pack has 5 foreskin pieces.

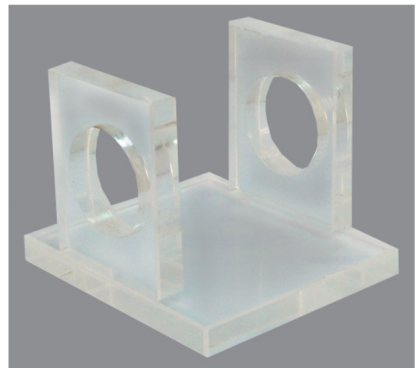


**GM-TH0001**

**Circumcision Holder**

The holder is made of clear Plexiglas and is used to hold a circumcision model for practicing circumcision.

# GINKGOMED COMPANY

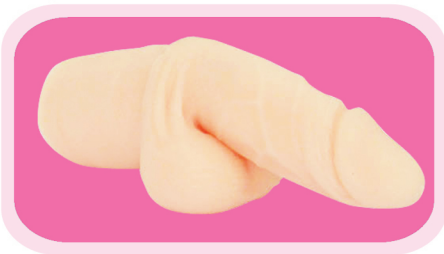

Tel: +886-2-2704-1032

Fax: +886-2-2704-0645

<http://www.ginkgomed.com.tw>

E-mail: [info@ginkgomed.com.tw](mailto:info@ginkgomed.com.tw)

11-1, No.5, Sec.2, An-He Road, Da-An District, Taipei City (106), Taiwan