

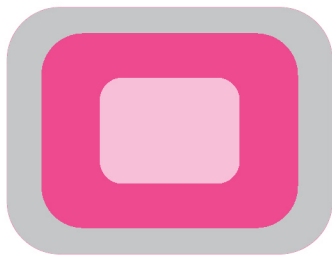
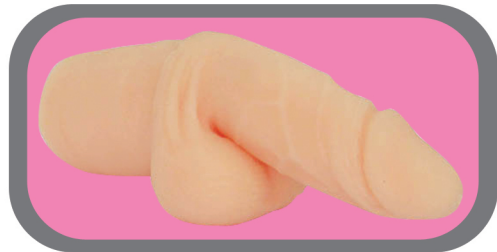


# CLINICAL TRAINING AIDS

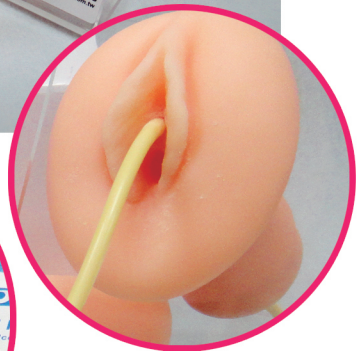

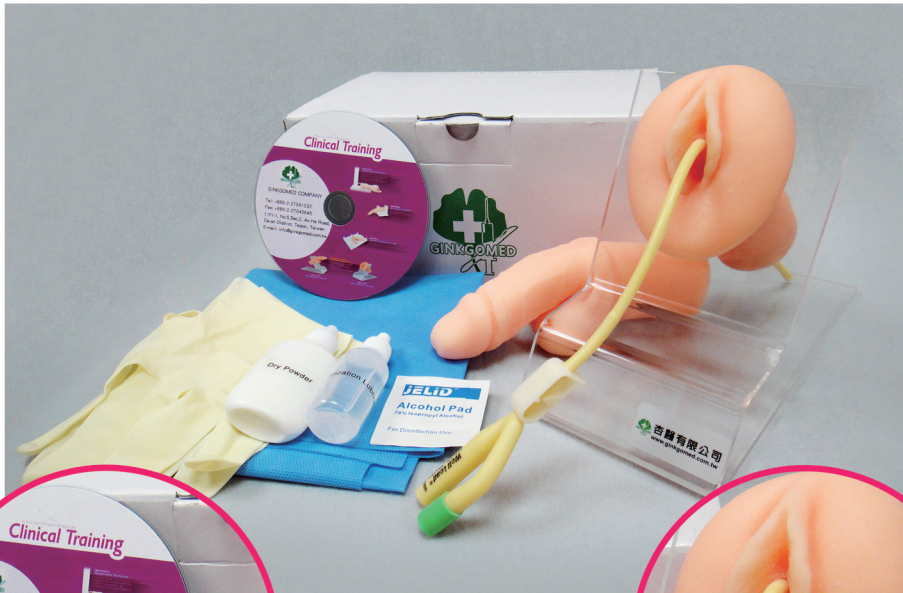

**GM-UC0010  
Dual Catheterization Training Set**

This training set is designed to make medical professionals to be skillful in doing urinary catheterization on male and female patients easily and quickly through repeated practices. Each set contains one male catheterization model, one female catheterization model, one Plexiglas holder and a pack of necessary accessories to start practice. Come with a disc of demonstration video.

**Content of Accessories:**


1. Disposable glove (1)
2. Dry powder (1)
3. Alcohol pad (1)
4. Fr-16 urinary catheter (1)
5. Catheterization lubricant (1)

## Urethral Catheterization


The model is used for practicing the procedure of urethral catheterization.

### How to use:

1. a. Prepare a catheterization set, a 16 - 18 Fr. Foley catheter, and disinfectant.
- b. Mount the model onto a stand.

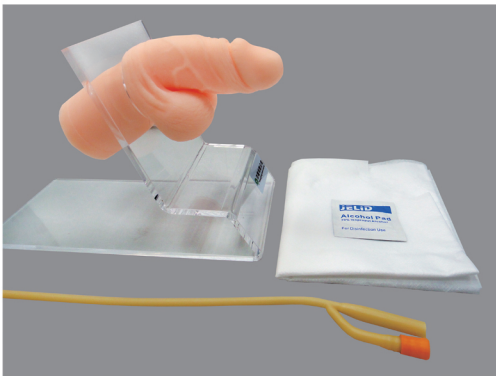

2. Disinfecting and draping the field.

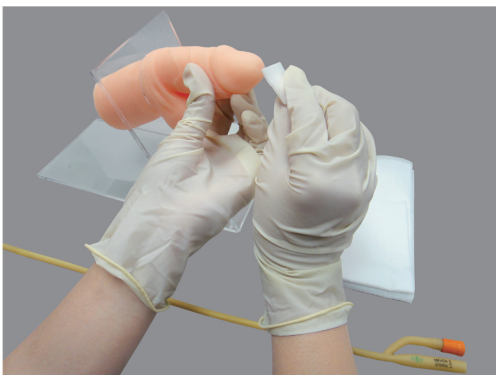

3. Check the catheter balloon by injecting a few ml of water.



4. a. Apply sufficient water soluble lubricant to the catheter.
- b. Insert the catheter until the tip of the catheter is completely inside the urinary bladder.
- c. Inflate the balloon of the catheter with 5 – 10 cc of water.
- d. Adjust the position of the catheter by withdrawing the catheter.



## Urethral Catheterization


The model is used for practicing the procedure of urethral catheterization.

### How to use

1. a. Prepare a catheterization set, a 16 - 18 Fr. Foley catheter, and disinfectant.
- b. Mount the model onto a stand.



2. Disinfecting and draping the field.

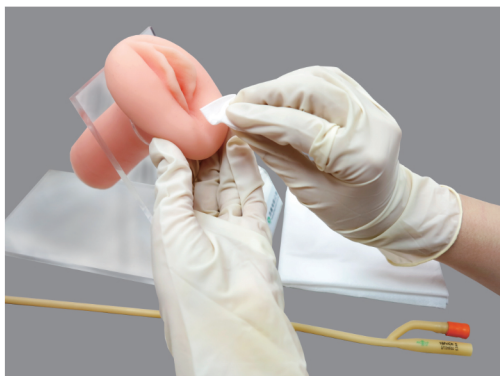


3. Check the catheter balloon by injecting a few ml of water.



4. a. Apply sufficient water soluble lubricant to the catheter.
- b. Insert the catheter until the tip of the catheter is completely inside the urinary bladder.
- c. Inflate the balloon of the catheter with 5 – 10 cc of water.
- d. Adjust the position of the catheter by withdrawing the catheter.



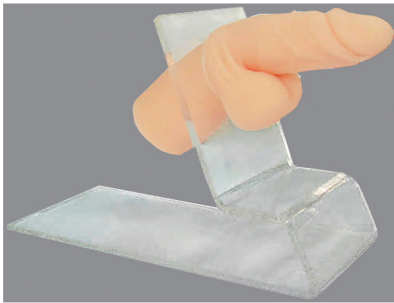

**GM-UC0020**

**Male Catheterization Training Set**

It is similar to GM-UC0010 with the same content of accessories but is used for practicing urinary catheterization on male patient only. Come with a disc of demonstration video.



**GM-UC0030**

**Female Catheterization Training Set**


It is similar to GM-UC0010 with the same content of accessories but is used for practicing urinary catheterization on female patient only. Come with a disc of demonstration video.



**GM-UC0021**

**Male Catheterization Model**

The model is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable. It is suitable for Fr.12-16 Foley catheters and is durable for repeated male urinary catheterization.

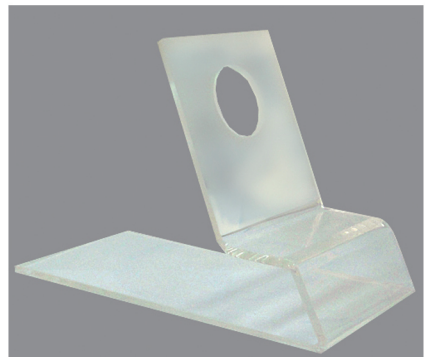

**GM-TH0002**

**Catheterization Holder**

It is made of clear Plexiglas and is used to hold the male or female catheterization model in place for urinary catheterization.




**GM-UC0031**

**Female Catheterization Model**

The model is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable. It is suitable for Fr.12-16 Foley catheters and is durable for repeated female urinary catheterization.

# GINKGOMED COMPANY

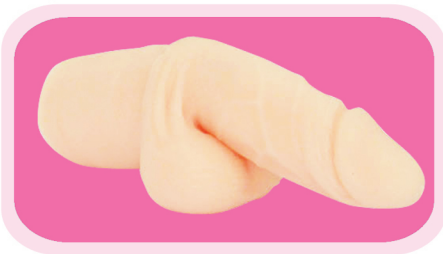

Tel: +886-2-2704-1032

Fax: +886-2-2704-0645

<http://www.ginkgomed.com.tw>

E-mail: [info@ginkgomed.com.tw](mailto:info@ginkgomed.com.tw)

11-1, No.5, Sec.2, An-He Road, Da-An District, Taipei City (106), Taiwan