



CLINICAL TRAINING AIDS

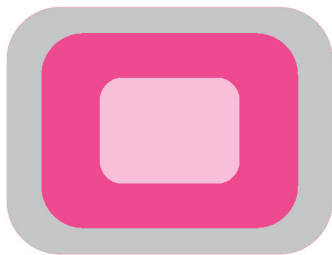

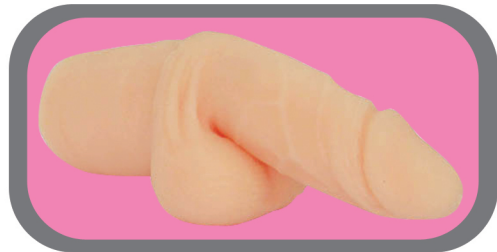

GM-HV0010


Venipuncture Training

This training set is designed to make medical professionals to be skillful in doing venous injection or blood withdrawal from a vein in the hand easily and quickly through repeated practices. Each set contains one adult or child TPR hand model, one Plexiglas holder and a pack of necessary accessories to start practice. The hand model has a vein-like tunnel vein in the dorsal side. It allows fluid infusion from the cylinder of the holder and is visible once a blood-like colored fluid is used. The nature of TPR model gives user a realistic tactile feeling of the hand and is durable for repeated needle punctures. Come with a disc of demonstration video.

Content of Accessories:

1. Disposable glove (1)
2. Dry powder (1)
3. Alcohol pad (1)
4. Blood-simulated powder (1)
5. 5 ml syringe with #23 needle (1)


Instruction For Intravenous Model

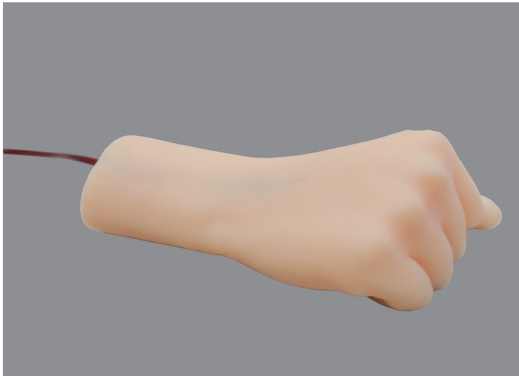

This a hand mode mad from synthetic material, designed for practicing intravenous injection andavenous puncture. The students can also rehearse how to prepare intravenous drugs and fluid, aseptic techniques, disposal of used needles and syringes, etc.

How to use:


1. connect the holder stand and the hand model with asuitable tube.
2. fill holder the stand with simulated blood (red water)
3. Remove the air inside the vein by an empty syringe if needed.
4. The vein is now visible and can be palpated easily.
5. The following procedures can be practice:
 - Disinfection and other aseptic techniques if needed.
 - Intravenous puncture and blood drawing.
 - Intravenous drug administration.
 - Adequate management of used needles and syringes.

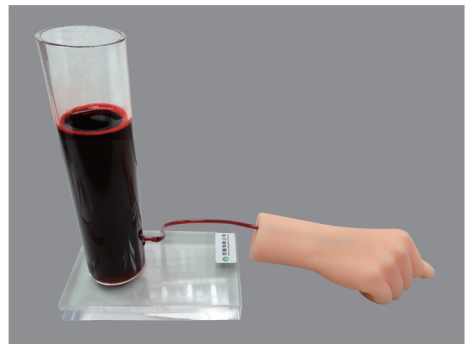



GM-HV0011
Adult Venepuncture Hand Model

The model is made of thermo plastic rubber (TPR), no heavy metal and toxic components detectable. It has shape and size of an adult hand with a vein-like tunnel vein in the dorsal side for repeated needle punctures.

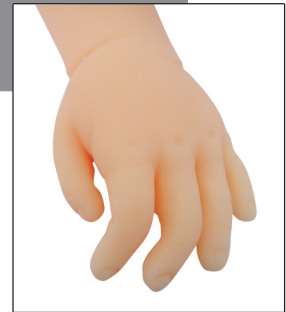

GM-HV0012
Child Venepuncture Hand Model

It is similar to the GM-HV0011 but with shape and size of a child hand.




GM-TH0005
Venepuncture Holder

It is made of clear Plexiglas with a stand base and a cylinder.

The cylinder can be connected to the vein-like tunnel vein of the hand model and is used for filling solution of blood-simulated powder.

GINKGOMED COMPANY

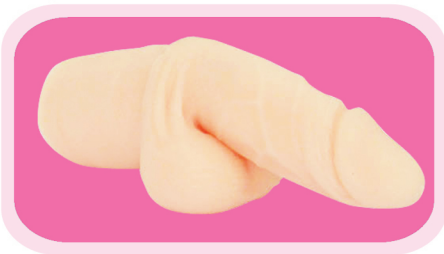

Tel: +886-2-2704-1032

Fax: +886-2-2704-0645

<http://www.ginkgomed.com.tw>

E-mail: info@ginkgomed.com.tw

11-1, No.5, Sec.2, An-He Road, Da-An District, Taipei City (106), Taiwan